GERSON®

3230 - Extreme Comfort™ N95 RESPIRATOR

NIOSH Approved: TC-84A-9285, MADE IN U.S.A.

3230 Individually Packaged

The Emergency Use Authorization (EUA) issued by the Food and Drug Administration (FDA) on March 2, 2020 permits use of this NIOSH certified respirator in medical settings to mitigate transmission of COVID-19*.

- Up to 50% lower breathing resistance compared to many molded N95 designs.
- Durable outer layer and super-soft liner - Made to withstand hardworking environments while providing all day comfort.
- Folded, pouch style design for easy carrying and compact storage.
- Individually packaged to help maintain cleanliness and reduce the risk of contamination.
- Synthetic rubber head straps designed for a comfortable and secure tension profile and extended life. Not made with natural rubber latex.
- Staple-free construction for comfort and safety.
- Adjustable enclosed nosepiece helps reduce potential fogging of eyewear.
- Meets CDC Guidelines for TB Exposure Control.
- Fluid Resistant – 120mm HG.

PACKAGING

Part Number	Packaging	Case Weight & Dimensions	Case Cube	Inner Box Bar Code	Case Bar Code
083230	50/box, 4 Bx/Case (200 Pcs.), 80 cs/Pallet	6 lbs - 9" x 12" x 14"	.9 cf	0 44585 32301 7	500 44585 3230

Respirator Size: 9.25" X 3.5".

*WARNINGS: No respirator can eliminate risk of contracting any disease. Do not use in or around sandblasting. Read and Understand Instructions and Warnings before using this product.

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1. Unwrap respirator and bend nosepiece into a curve to open mask.
2. Separate the pre-scored elastic headband into two individual bands.
3. Hold the headbands with your index fingers and thumbs.
4. Cup the open mask **securely** under your chin.
5. Pull both headbands over your head.
6. Rest the top headband on the crown of your head and bring the bottom one down under your ears.
7. Press the nosepiece around the contour of your nose and cheekbones. Adjust the edges of the mask to make sure there are no gaps.
8. Perform a Fit Check by gently cupping both hands around the mask, then exhale sharply to see if you sense or detect any leaks.

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WARNING: A respirator does not eliminate the risk of contracting any disease or infection. Occupational use of this respirator requires that you be fit tested by a qualitative or quantitative fit test and that your employer comply with OSHA 29 CFR 1910.134, which includes medical evaluation, training and fit testing.

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WARNINGS

Read and understand warnings on insert before using.

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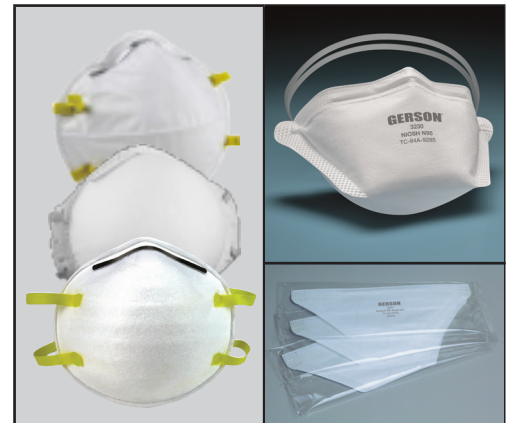
100% polypropylene
100% polypropylène
100% polipropileno

Molded vs. Foldable Comparison Chart

MOLDED

Typical Molded N95
(non-valved)

FOLDABLE

3230 Foldable N95
(non-valved)

Protection	N95	✓ NIOSH Approved Various	✓ NIOSH Approved TC-84A-9285
	Approved for Medical Use	X / ✓	✓
Construction	Durable outer shell	✓	✓
	Low profile, provides optimal field of vision	✓	✓
	Staple free construction	X / ✓	✓
	Nosewire enclosed	X	✓
Comfort	Comfort Level	Moderate	Extreme Comfort
	Up to 50% lower breathing resistance compared to many molded N95 designs.	X	✓
	Extremely reduces eyewear fogging	X	✓
Convenience	Folded, pouch style design for easy carrying and compact storage	X	✓
	Individually packaged to help maintain cleanliness and reduce the risk of contamination.	X	✓
	Easy to carry	X	✓
	Vending-Ready	X	Coming in 2021

GERSON

16 Commerce Blvd.

Middleboro, MA 02346

Tel 508.947.4000 Fax 508.947.5442

sales@gersonco.com • www.gersonco.com


© 2021 Louis M. Gerson Co., Inc. All Rights Reserved

Molded vs. Foldable Testimonials

Testimonials

MOLDED	VS.	FOLDABLE
<i>"Molded N95s are what I used to wear. They are effective BUT..."</i>		<i>"I tried Gerson's Foldable N95! This is the mask I will be wearing in 2021 because..."</i>
<i>"I get face fatigue"</i>		<i>"it is extremely comfortable"</i>
<i>"the straps can be tight"</i>		<i>"the straps are great!"</i>
<i>"the straps are a problem"</i>		<i>"the straps have a nice give and take balance"</i>
<i>"it is difficult to wear all day"</i>		<i>"I can wear it all day - no problem!"</i>
<i>"the dome easily crushes - can't take it anywhere"</i>		<i>"before I need to use it, the folded mask fits in my pocket"</i>

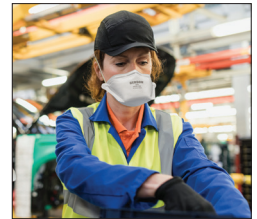GERSON®

16 Commerce Blvd.
Middleboro, MA 02346
Tel 508.947.4000 Fax 508.947.5442
sales@gersonco.com • www.gersonco.com
© 2021 Louis M. Gerson Co., Inc. All Rights Reserved